

## Bonding Agent Technique - Option 1: Grind then Soak Teeth



- 1** Remove all wax from teeth prior to grinding. (Preheat hot plate. Set temperature to 100\* and allow at least 10 minutes to preheat.)




- 2** Deglaze the ridge lap and grind-in teeth based on the vertical dimension.



- 3** Place the prepared and cleaned teeth into the metal container kit with Eclipse Bonding Agent liquid as indicated in the Directions For Use.



- 4** Place the metal container inside the metal ring on the hot plate. Temperature should already be set to 100\* (from step 1). The rectangle for 40°C on the temperature strip should turn GREEN between 4 to 6 minutes. (May have to adjust temperature dial slightly.)

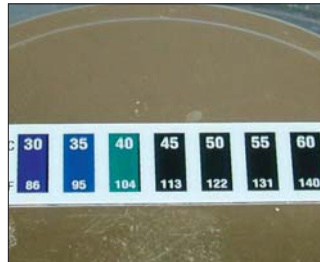

- 5** Remove teeth with tweezers from the bonding agent immediately and allow them to dry before proceeding to set-up the teeth and contour with Eclipse Resin.

\*For hot plates with a digital temperature reading, set the temperature to 90.

NOTE: If more than 1.5 mm of any tooth is ground off of the ridge lap area after the bonding agent application, the tooth will need bonding agent reapplied.

## Bonding Agent Technique - Option 2: Soak then Grind Teeth



**1** Remove all wax from teeth. (Preheat hot plate. Set temperature to 100\* and allow at least 10 minutes to preheat.)




**2** Deglaze the ridge lap.



**3** Place the deglazed and cleaned teeth into the metal container kit with Eclipse Bonding Agent liquid as indicated in the Directions For Use.



**4** Place the metal container inside the metal ring on the hot plate. Temperature should already be set to 100\* (from step 1). The rectangle for 40°C on the temperature strip should turn GREEN between 4 to 6 minutes. (May have to adjust temperature dial slightly.)

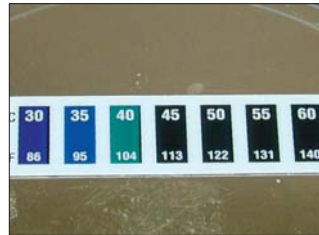

\*For hot plates with a digital temperature reading, set the temperature to 90.



**5** Remove teeth with tweezers from the bonding agent immediately and allow them to dry.



**6** Minimal grinding of the tooth ridge lap area can be done. Set-up teeth and contour using Eclipse resin.

NOTE: If more than 1.5 mm of any tooth is ground off of the ridge lap area after the bonding agent application, the tooth will need bonding agent reapplied.